WELLSACLEAN A

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REF WCC705

Produktbeschreibung

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REF WCC705

Product description

WELLSACLEAN A ABDRUCKDESINFEKTION is a concentrate based on quarternary ammonium compound and amine for preparation of a working solution for manual disinfection of invasive medical devices like dental impressions in an immersion bath. The product is free of aldehydes and phenol.

For hygienic reasons, dental impressions should be disinfected in the dental practice before being sent to dental laboratories in order to avoid infections. The disinfection procedure should be repeated by the dental laboratories before starting to work with the dental impressions.

Area of application

For disinfection of dental impressions made from silicones, polyether, and hydrocolloids (alginates).

Advice for users

Application only by qualified personnel.

Do not mix with other products.

Avoid contact with styptic medication.

Comply exactly with the recommended contact time.

Clean immersion bath and application aids thoroughly prior to the first application of the product, after application of other products, and at the end of the working day.

Report serious incidents related to this product to the manufacturer and the respective authorities. The safety datasheet is available at www.wellsamed.com.

Application

Put on protective clothing, eye protection, face mask, and protective gloves before each disinfection!

Disinfection in an immersion bath:

1. Prepare a 6% working solution: Provide 940 ml water (approx. 20 °C, drinking water quality), add 60 ml concentrate and mix.
2. Place the dental impression including the tray in the working solution directly after the impression was taken. Move the impression tray back and forth in order to avoid air bubbles.
3. After the contact time has elapsed: Rinse dental impression including the impression tray under running water (approx. 20 °C, drinking water quality).

Comply with the instrument manufacturer's recommendations for reprocessing as well as with national requirements for reprocessing of dental impressions and impression trays.

Shelf life and expiry date

Shelf life of working solution: 1 day. Renew immediately in case of visible contamination.

See product label for expiry date. Do not use the product after the expiry date.

For further information on the reprocessing of medical devices, recommendations are available online on the homepage of the Robert Koch-Institute (Recommendation from the Commission on Hospital Hygiene and Infection Protection) at the Robert Koch Institute (RKI) on the „Hygiene requirements for the reprocessing of medical devices“.

For efficacy spectrum and information on material compatibility see page 2.

WELLSACLEAN A

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REF WCC705

Efficacy spectrum/concentration/contact times

Efficacy under dirty conditions (assessed at room temperature)	standard	concentration	contact time
BACTERIA			
P. aeruginosa, S. aureus, E. hirae	EN 13727	6 %	15 min
	EN 14561		
YEASTS			
C. albicans	EN 13624	6 %	15 min
	EN 14562		
ENVELOPED VIRUSES			
MVA Modified Vacciniavirus Ankara	EN 14476	6 %	15 min

Material compatibility

Impression Material	silicone, polyether (polysulfone), hydrocolloids (alginates)
Metals	stainless steel
Plastics	PE, PSU, PVC
Rubber, flexible Plastics	silicone
Others	ceramics
Advice	Upon compliance to the intended use as well as the indicated contact time and concentration, no material damage is to be expected with the above mentioned materials. Avoid contact of more than 48 hours with polycarbonates (Makrolon). Test material compatibility for polyacrylates and polycarbonates before use. For copper, brass, and aluminium a colour change is possible upon prolonged contact time. Natural rubber and EPDM may lose elasticity and change their colour upon prolonged contact time. Test material compatibility for not mentioned materials.

Storage advice

Store product between 5 and 25 °C. Keep the container in an upright position in a well-ventilated place. Keep container tightly closed. Keep only in original container. Keep away from food, drink, and animal feedingstuffs.

Composition

Active ingredients in 100 g of concentrate:
 3.15 g Didecyloxyethylmethylammonium propionate,
 2.50 g N,N-Bis(3-aminoethanol)dodecylamine
 2.00 g Cocospropylendiamindiguanidinium acetate

Physico-chemical data

Appearance of concentrate: clear, blue liquid
 pH value: 7.2 – 8.0
 rel. density: 0.990 – 0.999 g/cm³

Labelling according to Regulation (EC) No. 1272/2008 (CLP)

For information on labelling according to Regulation (EC) No 1272/2008 (CLP) see safety datasheet and label.

Labelling

CE 0123 **MD** IIb

Expert opinions & further information

WELLSACLEAN A ABDRUCKDESINFEKTION has been assessed by independent laboratories.

The listings, reports, and notes in this product information refer to the product PLIWA Biguaton Spezial, which is identical in formulation.

For further details concerning expert opinions, information on products, and answers to specific questions contact us by e-mail (info@wellsamed.com).

The manufacturer has a manufacturing authorisation according as per §13 German medicines act as well as a GMP- and an EN 13485 certificate.

 PLIWA Hygiene GmbH,
 Pliwastraße 2, 34323 Malsfeld, Germany

Distributed by:
 wellsamed GmbH
 Brahestraße 16, 04347 Leipzig, Germany